
Mölnlycke®

Mesalt®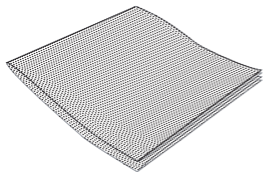

CE 2797

Manufacturer / Fabricant

Mölnlycke Health Care AB  
Gamlestadvägen 3C, Box 13080  
SE-402 52 Göteborg, Sweden

[www.molnlycke.com](http://www.molnlycke.com)

Single use  
Usage unique

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**sr** Ne koristiti ako je omot oštećen

**Intended use**

Management of moderate to heavily exuding non-infected/infected wounds such as pressure ulcers, foot ulcers and deep wound management e.g. cavity wounds and fistulas.

**Mode of action**

Mesalt consists of a soft viscose/polyester nonwoven material impregnated with sodium chloride. Mesalt absorbs wound exudate and the sodium chloride is released. Mesalt becomes hypertonic, thereby exudate, bacteria and necrotic material is drawn into the dressing, facilitating the natural wound healing process.

**Instruction for use**

Application:

1. Clean the wound area and debride of any excessive necrotic tissue or eschar.
2. Dry the surrounding skin, but do not dry the surface of the wound.
3. Where clinically indicated, a water-resistant cream or ointment can be used on the surrounding skin.
4. Choose the appropriate Mesalt dressing size. For shallow wounds do not allow Mesalt to overlap the wound edges. For deep wounds, loosely fill the wound.
5. Apply Mesalt dry into the wound
6. Depending on the level of exudate, cover with an appropriate secondary dressing.
7. Where clinically indicated, compression therapy may be used in conjunction with Mesalt.

Mesalt is intended for short-term repeated use.

Pre-clinical testing supports use up to 30 days.

**Dressing change and removal**

Mesalt should be changed when saturated (at least once a day), or according to good clinical practice. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional.

**Precautions**

- Mesalt should not be used on patients with a known allergy to the dressing or its components.
- Mesalt should not be used on dry wounds.
- Mesalt should not be allowed to come into direct contact with exposed bone or tendon.
- Infected wounds should be evaluated on a regular basis and treated appropriately.

**Storage and disposal**

Mesalt should be stored in dry conditions (below 60%RH) and below 25°C (77°F).

Disposal should be handled according to local environmental procedures.

**Other information**

- Mesalt contains 20% w/w sodium chloride.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package is damaged or opened prior to use. Do not re-sterilise.
- If the product is used after the expiry date, product properties cannot be ensured.

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